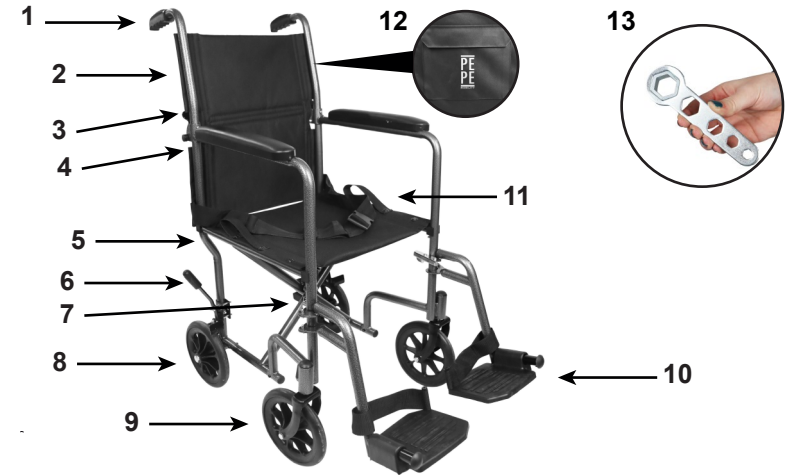
**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE**

**TRANSPORT WHEELCHAIR (NARROW)
SILLA DE RUEDAS TRANSPORTE (ESTRECHA)
TRANSPORTROLLSTUHL (SCHMAL)
FAUTEUIL ROULANT DE TRANSPORT (ÉTROIT)
CARROZZINA DA TRASPORTO (STRETTA)**

1 COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO
1.	Handle	Empuñadura	Handgriff	Poignée	Maniglia
2.	Backrest	Respaldo	Rückenlehne	Dossier	Schienale
3.	Folding tab	Pestaña de plegado	Klappbare Lasche	Languette rabattable	Linguetta pieghevole
4.	Armrest	Reposabrazos	Armlehne	Accoudoir	Bracciolo
5.	Seat	Asiento	Sitz	Siège	Sedile
6.	Brake lever	Palanca de freno	Bremshebel	Levier de frein	Leva del freno
7.	Assist lever	Palanca de ayuda	Assistenzhebel	Levier d'assistance	Leva di assistenza
8.	Rear wheel	Rueda trasera	Hinterrad	Roue arrière	Ruota posteriore
9.	Front wheel	Rueda delantera	Vorderrad	Roue avant	Ruota anteriore
10.	Footrest	Reposapiés	Fußstütze	Repose-pieds	Poggiapiedi
11.	Belt	Cinturón	Gurt	Ceinture	Cintura
12.	Pocket	Bolsillo	Tasche	Poche	Tasca
13.	Fixed key	Llave fija	Fester Schlüssel	Clé fixe	Chiave fissa

2

WIDTH	HEIGHT	LENGTH	FOLDING DIMENSIONS
54 cm (21")	100 cm (39")	98 cm (38.5")	29x74x59 cm (11.5"x29"x23")

SEAT HEIGHT	SEAT DIMENSIONS	BACKREST HEIGHT	Ø WHEELS
48 cm (19")	44x40 cm (17"x15.5")	48 cm (19")	20 cm (7.8")

MAX. USER WEIGHT	NET WEIGHT	TOTAL WEIGHT (with footrest)
100 kg (220 lb)	11 kg (24.5 lb)	13 kg (28.5 lb)

EN The **PEPE - Transport Wheelchair (Narrow)** is a device designed to help transport people who are paraplegic, quadriplegic, hemiplegic or who, due to illness or accident, have to use a wheelchair temporarily. The user should not exceed the maximum recommended weight of 220 lb. The chair is designed for the user to be transported by a third person with sufficient strength to push the chair safely. It has a safety belt for greater user restraint.

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3 ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO

EN 1. **Unfold** the wheelchair by pulling the armrests outward.

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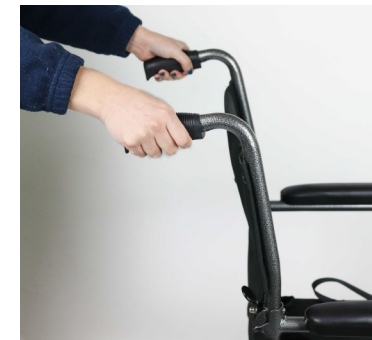
2

EN 2. **Lift the backrest upward** until it clicks into place.

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3

EN 3. **Insert the footrests** by fitting the holes and rotate it until the footrest is facing the front.

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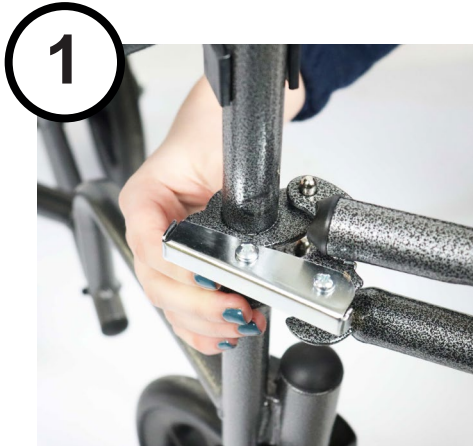
EN 1. To remove the footrests, use the metal lever to unclip the footrests and remove them from the chair frame.

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EN 2. Press the folding tab located at the back of the chair and fold the handles downward.

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EN 3. Pull the straps located on the seat upwards to fold the chair frame.

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EN 1. Adjusting the footrest. The footrest can be adjusted in height for more comfortable foot support.

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EN 2. Adjust the front wheels. With use, the front wheels may become loose and the fixed wrench helps to tighten them.

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EN The maximum slope for the use of parking brakes is 12° for a rear slope and 11° for a front slope.

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P10019

MATERIALS

- Structure: steel.
- Armrests, handles and plastic finishes: PVC.
- Fabric: fireproof nylon.

WARNINGS

- Do not use the wheelchair without carefully reading the instructions.
- This chair may not be used as a seat in vehicles of any kind.
- Always transfer the wheelchair user to a seat approved by the vehicle manufacturer and use the available safety devices.
- Do not use this wheelchair on streets, roads or any surface with vehicle traffic.
- Do not attempt to use the wheelchair on ramps without proper assistance.
- Before inserting or removing the user, the wheelchair must be on a smooth and stable surface.
- Do not apply weight on the footrests when inserting or lifting the user into or out of the wheelchair.
- Do not lean or drop weight on the top of the wheelchair backrest, it may cause the wheelchair to tip over.
- Do not stand on the wheelchair.
- Sudden changes in direction when using a ramp or slope can cause instability and/or the possibility of tipping.
- Keep in mind that this chair is intended for transporting another person, and not for personal transport.

- Never stretch so far that you have to move around in the seat or out of the chair.
- When you have to reach for objects that require you to lean, make sure the front wheels point forward, as this will stabilize the chair.
- The temperature of the wheelchair may increase when exposed for long periods of time to external heat sources, such as sunlight.
- If you will be storing the wheelchair for an extended period of time, do not leave it exposed to prolonged weathering.
- Do not exceed the maximum supported weight, which is 220 lb.
- In accordance with the manufacturer's recommendations, this product is suitable for recycling at designated recycling points.
- Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Retain these instructions for future reference.

MAINTENANCE

- Inspect the chair before each use to ensure that there is no damage or excessive wear. If you find any signs of wear or damage to your chair, DO NOT USE IT.
- Regularly check that all parts are sound before operation and that the casters turn smoothly. If you notice that the casters are loose, tighten them using the wrench provided.
- Keep the wheelchair clean, the steel tubes can be easily cleaned with a damp cloth. The fabric parts should be washed by hand (max. 30°) and with neutral soap.

WARRANTY

- KMINA warrants this wheelchair to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels and handgrips. This warranty does not cover:
- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
 - Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
 - KMINA is not liable for accidental damage or damage to property.

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